LECONSA

HOME APPLIANCE

ALCOHOL TESTER
ALKOHOLTESTER
ALKOHOLTESTER
ALKOHOLMÄTARE
ALKOHOLTESTER
ALCOHOLÍMETRO
L'ÉTHYLOMÈTRE
ALCOHOLTESTER
ALKOMAT

[Item no. 40293]

GB / DK / NO / SE / DE / ES / FR / NL / PL

DIGITAL ALCOHOL TESTER

[TECHNICAL DATA]

Net weight:	About 22.5 grams
Sensor:	High-sensitivity direct heating wind-sensitive sensor
Test Range:	0.00-0.19% BAC (0.0-1.9g/l)
Charging voltage:	DC 5V
Warm-up time:	10 seconds
Blowing time:	Within 10 seconds
Response time:	< 5 seconds
Automatic shutdown:	10 seconds after the result is displayed
Product size:	89x24.5x17mm
Working environment:	0°C-45°C

IN ORDER FOR YOU TO USE THIS PRODUCT BETTER,
PLEASE READ THE INSTRUCTIONS CAREFULLY AND
STRICTLY FOLLOW THE INSTRUCTIONS BEFORE USE.

PRODUCT OVERVIEW

1. Blow nozzle
2. Indicator light
3. Display screen
4. Switch key
5. USB charging port

HOW TO USE

Warm up:

1. Press and hold the power button for 2 seconds, after a beep, the LCD screen of the tester will be activated.
2. The "warm up" character appears on the display, and the number counts down from 10 to 00.

Notice:

Since the sensor surface will be oxidized when the tester is not used for a long time, when the tester is taken out of the package for the first time or the tester has not been used for a long time, the user needs to preheat the product as many times as possible before performing the alcohol test.

Test Methods:

1. After the "preheating" is completed, it enters the "blowing" state, the display shows the blowing character, and the number starts counting down from 10 to 00.
2. Blow at any time within this countdown. After the machine senses the gas, the machine counts down for 3 seconds, and the buzzer will beep to stop blowing.
3. Read the test result from the display of the tester.
4. If the result of the alcohol concentration is 0.00% BAC (or 0.0g/l), it means that there is no drunk driving and the green indicator light is on.

5. If the alcohol concentration is within the display range below the result, the "warning" character will appear on the display, accompanied by a beep, and the yellow indicator light will light up. Alcohol concentration $>0.01\%$ BAC (or 0.1g/l); The alcohol concentration is less than 0.08% BAC (or 0.8g/l).
6. If the result of the alcohol concentration is in the following display range, the "Danger" character will appear on the display, accompanied by a rapid beep, and the red indicator light will light up. The alcohol concentration is $>0.08\%$ BAC (or 0.8g/l).

Precautions:

1. Test alter drinking for at least 20 minutes, because it takes about 20 minutes for alcohol to be absorbed from the digestive system into the blood. Immediately alter drinking, the results can only reflect the alcohol concentration in the mouth at that time, not the alcohol concentration in the blood.
2. In order to make the test result accurate, please wait 20 minutes alter drinking alcohol before performing the test. The test time is preferably more than 3 minutes from the last test.
3. Do not directly wipe the product with chemicals or solvents containing corrosive properties to prevent the liquid from flowing into the blowing port.
4. When there is a high concentration of interfering gas, the machine will not work normally.

- If the test is performed under low power, the test value will have a certain deviation from the actual value.
- Unauthorized repair and damage to internal parts will cause product failure.
- In order to achieve more accurate test results, it is recommended that users repeat the test 2 to 3 times.

COMMON FAULTS AND SOLUTIONS

FAILURE PHENOMENON	POSSIBLE CAUSES OF FAILURE	SOLUTION
Can't turn on	Very low battery	Please charge the product
No response when blowing	Not fully warmed up	
No result are displayed when blowing	There is residual gas in the product	Reheat the product
	Did not blow within the specified time	Blow within the time that the blow character is displayed
Results without blowing There is res	There is residual gas in the product	Warm up a few more times before testing
Several test results are different	Test immediately after drinking	Test 20 min. after drinking
	The blowing time is different	It's recommended to blow 3-5 seconds

ALCOHOL CONCENTRATION TABLE

BAC				BrAC	
mg/100ml	%	‰	g/l	mg/l	ppm
10	0.01	0.1	0.1	0.05	25
20	0.02	0.2	0.2	0.10	50
30	0.03	0.3	0.3	0.15	75
40	0.04	0.4	0.4	0.20	100
50	0.05	0.5	0.5	0.25	125
60	0.06	0.6	0.6	0.30	150
70	0.07	0.7	0.7	0.35	175
80	0.08	0.8	0.8	0.40	200
90	0.09	0.9	0.9	0.45	225
100	0.10	1.0	1.0	0.50	250
110	0.11	1.1	1.1	0.55	275
120	0.12	1.2	1.2	0.60	300
130	0.13	1.3	1.3	0.65	325
140	0.14	1.4	1.4	0.70	350
150	0.15	1.5	1.5	0.75	375
160	0.16	1.6	1.6	0.80	400
170	0.17	1.7	1.7	0.85	425
180	0.18	1.8	1.8	0.90	450
190	0.19	1.9	1.9	0.95	475
200	0.20	2.0	2.0	1.00	500

BAC: Blood Alcohol Concentration

BrAC: Breath Alcohol Concentration 0.05%

BAC: Means that every 100ml of blood contains 0.05g (50mg) alcohol

Commonly used units in various countries: mg/100ml-China %SAC-America g/l-Europe mg/L BrAC-Japan

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